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*Self-catheterisation with
UROMED SIMPLYCATH*

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Disposable Transurethral Catheter for Intermittent Self-catheterisation

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The introducer tip of SIMPLYCATH: intelligent protection against infection

The risk of cystitis is significantly lower with intermittent self-catheterisation (ISC) compared to the use of a permanent catheter. Nonetheless, the regular insertion of a catheter into the bladder runs the risk of introducing pathogenic environmental germs, causing urinary tract infections (UTI). Immediate and thorough treatment upon the appearance of any symptoms of inflammation is necessary to prevent complications, above all pyelonephritis. At the same time, it must be kept in mind that frequent antibiotic treatment promotes the development of resistance.

The use of UROMED SIMPLYCATH catheters helps to minimize the risk of urinary tract infection thanks to its integrated introducer tip. It works as a kind of protective sleeve which prevents the catheter from coming into contact with the distal 15 mm of the urethra, which is particularly full of germs.

	Charrière (Ch.) French (Fr.)	Durchmesser (Ø) Diameter (Ø)
	08	2,7 mm
	10	3,3 mm
	12	4,0 mm
	14	4,7 mm
	16	5,3 mm
	18	6,0 mm

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The Advantages of the UROMED SIMPLYCATH mobil Disposable Tran

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Only DEHP- and latex-free materials are used.

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The protection cap with ring and integrated nubs makes opening even more easy.

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The rounded, atraumatic eyes are the result of an innovative manufacturing process. They support a soft insertion of the catheter.

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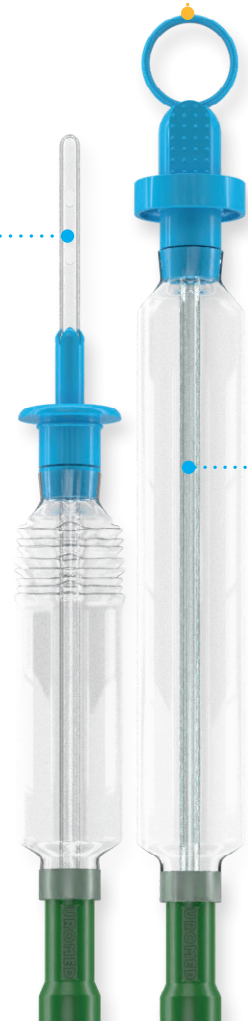
The silicone introducer tip helps to minimize the risk of urinary tract infections.

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The complete covering of the catheter with a specially developed, wafer-thin foil, enables a contact-free and at the same time gentle insertion of the catheter. Contamination through direct contact with the hand is thus avoided.

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
Transurethral Catheter at a Glance

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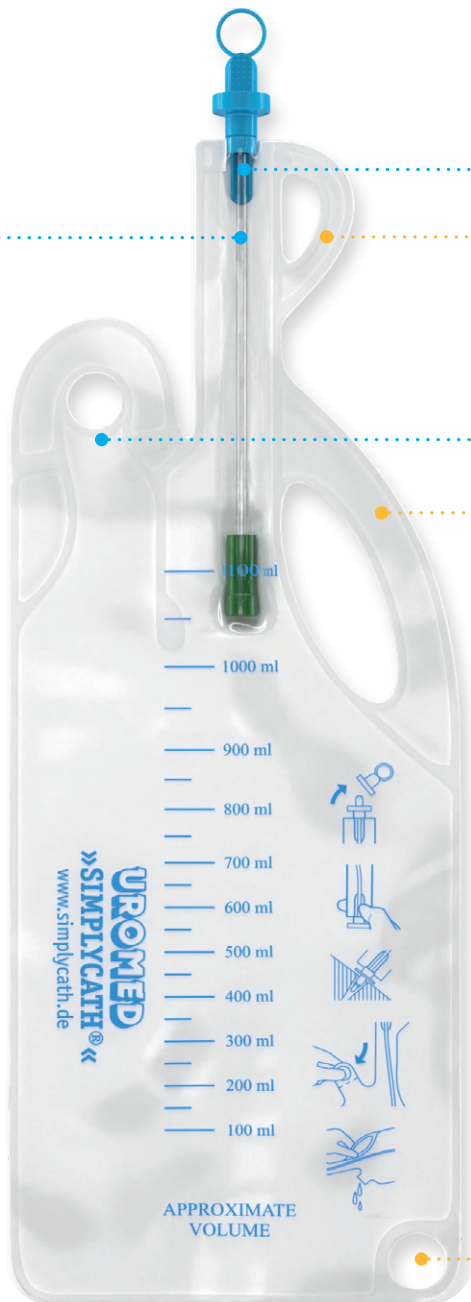The ready for use gel-coated catheter ensures high sliding capacity for gentle insertion and to prevent from injuries.

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The small grip enables an easy pushing forward of the catheter and guarantees more control while holding & pushing forward.

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Perforation of the bag facilitates the opening and allows a controlled release of urine.

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Holding loops are designed for best possible handling of the bag, especially for people with limited hand functions.

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A discreet application is given due to the opaque packaging and particularly low-noise foil.

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The UROMED SIMPLYCATH mobile series provides greater discretion and mobility – at home or on the move. The new, sophisticated product design is especially well suited to people with limited hand functions.

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Disposable catheter for intermittent self-catheterisation

- a protective cap with ring and nubs enables easy opening
- ready-to-use, gel-moistened catheter
- introducer tip for preventing the infiltration of germs
- rounded, atraumatic drainage eyes allow gentle insertion
- complete covering of the catheter with a thin foil facilitates a contact-free insertion
- thin foil with anti-slip-effect
- easy handling
- immediately usable
- small size
- opaque, low-noise packaging
- consists exclusively of DEHP and latex-free materials

REF REF	Ch./Fr. Ch./Fr.	Ø Ø	PZN CPN
3612.08	08	2,7 mm	11515138
3612.10	10	3,3 mm	11515144
3612.12	12	4,0 mm	11515150
3612.14	14	4,7 mm	11515167
3612.16	16	5,3 mm	11515196

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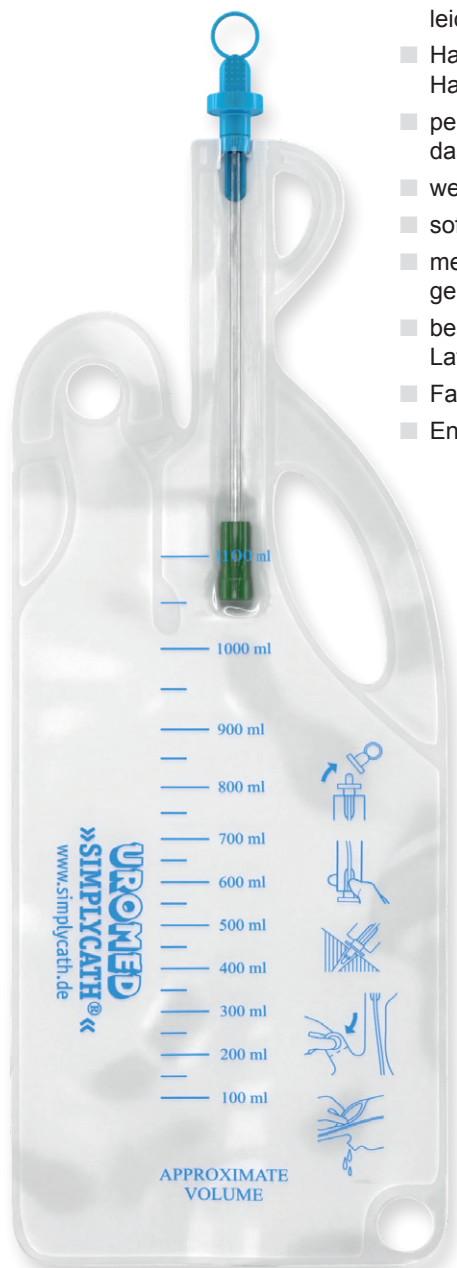
Disposable catheter for intermittent self-catheterisation

- a protective cap with ring and nubs enables easy opening
- ready-to-use, gel-moistened catheter
- introducer tip for preventing the infiltration of germs
- rounded, atraumatic drainage eyes allow gentle insertion
- the small grip enables an easier pushing forward of the catheter
- holding loops for optimal handling of the bag
- perforation of the bag facilitates the opening and allows a controlled release of urine
- soft bag material in bottle shape
- immediately usable
- opaque, low-noise packaging
- consists exclusively of DEHP and latex-free materials
- capacity: 1100 ml
- waste bag

REF REF	Ch./Fr. Ch./Fr.	Ø Ø	PZN CPN
3602.08	08	2,7 mm	11515084
3602.10	10	3,3 mm	11515090
3602.12	12	4,0 mm	11515109
3602.14	14	4,7 mm	11515115
3602.16	16	5,3 mm	11515121

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Disposable catheter for intermittent self-catheterisation

- ready-to-use, gel-moistened catheter
- introducer tip for preventing the infiltration of germs
- atraumatic drainage eyes
- thin foil with anti-slip-effect
- finger grip to fix the catheter
- perforation for tearing open the catheter covering and to enable urine-flow
- easy handling
- immediately usable

REF REF	Ch./Fr. Ch./Fr.	Ø Ø	PZN CPN
3610.08	08	2,7 mm	00230958
3610.10	10	3,3 mm	00230964
3610.12	12	4,0 mm	00230970
3610.14	14	4,7 mm	00232549
3610.16	16	5,3 mm	00232555

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

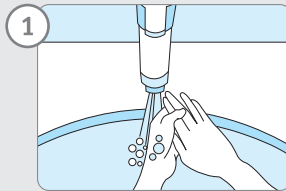
Wash your hands thoroughly before catheterisation. Then clean the genital area from the labia majora to the anus. Disinfect the urethral opening and the surrounding area with a mucosal disinfectant.

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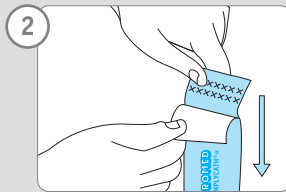
In order to open the packaging, pull downward on the flap of the top printed side. Remove the sterile catheter from the packaging.

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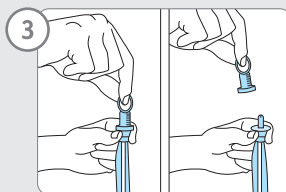
Now, remove the blue protective cap from the introducer tip while holding the catheter firmly at the blue plastic attachment.

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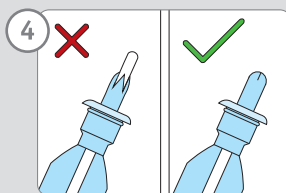
The catheter tip must not protrude out of the introducer tip to prevent the entry of germs into the urethra.

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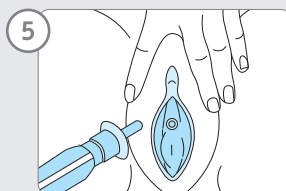
With a little practice in the right position, inserting the catheter becomes a simple matter. Assume a relaxed seated position to do so. Insert the introducer tip into the urethra until the disc reaches the urethral opening. Keep the introducer tip in this position.

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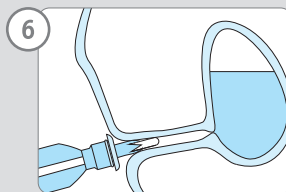
Then use one hand to fix the introducer tip in the urethra. Hold the catheter in your other hand and slide it onward slowly until the bladder has been reached and urine starts to flow.

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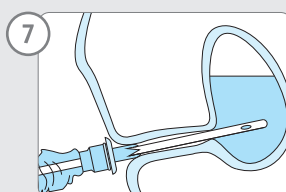Leave the catheter in this position until the urine flow stops. Gently pressing on the lower abdomen and slowly pulling out the catheter enables the residual, still remaining urine to also flow out of the bladder.

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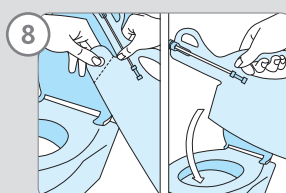Pull the catheter out of the urethra slowly and carefully. If you use the catheter with a urine bag, you can rip it open at the perforation to empty it. Dispose of the material in the household waste. Then wash your hands thoroughly once again.

UROMED SIMPLYCATH *mobil* transurethraler Einmalkatheter *Disposable Transurethral Catheter*

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*The UROMED SIMPLYCATH
mobile series provides greater
discretion and mobility –
at home or on the move.
The new, sophisticated prod-
uct design is especially well
suited to people with limited
hand functions.*

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Disposable catheter for intermittent self-catheterisation

- *a protective cap with ring and nubs
enables easy opening*
- *ready-to-use, gel-moistened catheter*
- *introducer tip for preventing the
infiltration of germs*
- *rounded, atraumatic drainage eyes
allow gentle insertion*
- *complete covering of the catheter
with a thin foil facilitates a contact-
free insertion*
- *thin foil with anti-slip-effect*
- *easy handling*
- *immediately usable*
- *opaque, low-noise packaging*
- *consists exclusively of DEHP and
latex-free materials*

REF REF	Ch./Fr. Ch./Fr.	Ø Ø	PZN CPN
3613.10	10	3,3 mm	15784786
3613.12	12	4,0 mm	15784792
3613.14	14	4,7 mm	15784800
3613.16	16	5,3 mm	15784817

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Disposable Transurethral Catheter with Integrated Urine Bag

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Disposable catheter for intermittent self-catheterisation

- *ready-to-use, gel-moistened catheter*
- *introducer tip for preventing the infiltration of germs*
- *atraumatic drainage eyes*
- *thin foil with anti-slip-effect*
- *finger grip to fix the catheter*
- *perforation for tearing open the catheter covering and to enable urine-flow*
- *easy handling*
- *immediately usable*

REF	Ch./Fr.	Ø	PZN
REF	Ch./Fr.	Ø	CPN
3600.10	10	3,3 mm	00230852
3600.12	12	4,0 mm	00230906
3600.14	14	4,7 mm	00230912
3600.16	16	5,3 mm	00230929
3600.18	18	6,0 mm	00230935

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Disposable catheter for intermittent self-catheterisation

- ready-to-use, gel-moistened catheter
- introducer tip for preventing the infiltration of germs
- atraumatic drainage eyes
- finger grip to fix the catheter
- holding loops for optimal handling of the bag
- peel off strap for draining the bag
- soft bag material in bottle shape
- easy handling
- immediately usable
- capacity: 1100 ml
- waterproof bolster
- waste bag

REF REF	Ch./Fr. Ch./Fr.	Ø Ø	PZN CPN
3601.08	08	2,7 mm	09949266
3601.10	10	3,3 mm	09949272
3601.12	12	4,0 mm	09949289
3601.14	14	4,7 mm	09949295
3601.16	16	5,3 mm	09949303
3601.18	18	6,0 mm	09949326

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
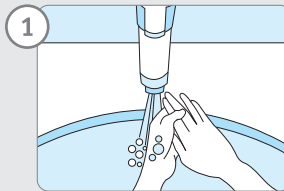
Application of the Disposable Transurethral Catheters for Men

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
Wash your hands thoroughly before catheterisation. Then clean the glans and then the urethral opening. Disinfect the urethral opening and the surrounding area with a mucosal disinfectant.

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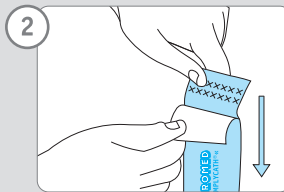
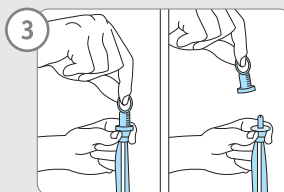
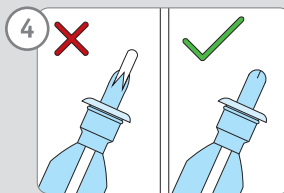
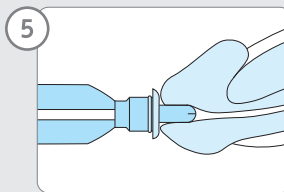
With a little practice in the right position, inserting the catheter becomes a simple matter. Assume a relaxed seated or standing position to do so. Insert the introducer tip into the urethra until the disc reaches the urethral opening. Keep the introducer tip in this position.

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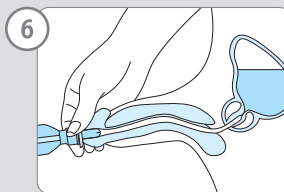
Use one hand to fix the introducer tip in the urethra and stretch the penis out while doing so. Hold the catheter in your other hand and slide it onward slowly until the bladder has been reached and urine starts to flow.

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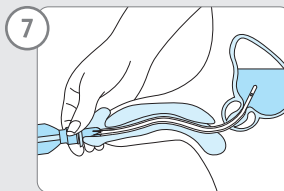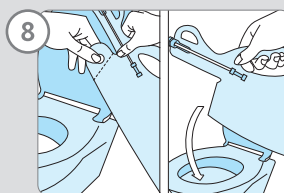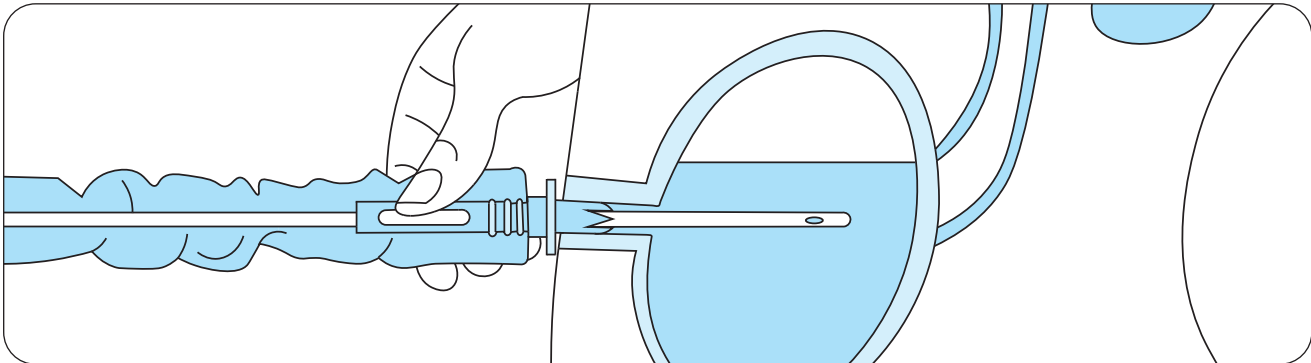
Neobladder with navel connection

For catheterisation via navel connection, we recommend the UROMED SIMPLYCATH mobil REF 3613 disposable transurethral catheter for convenient drainage into the toilet or a disposable transurethral catheter with integrated REF 3601 urine bag.

Our empirical values indicate that large-diameter catheters are preferable due to the usually increased mucous production of small intestinal tissue, since they provide higher drainage performance.

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Neobladder with urethral connection

For catheterisation via urethra, please be so kind as to recommend to your patients the UROMED SIMPLYCATH disposable transurethral catheters for women, shown on pages 6 and 8 or the version for men, shown on pages 10 and 12.

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For more information

If you are interested or in case of open questions regarding our products and services, please feel free to contact us.

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©UROMED Kurt Drews KG