UROMED
Stonizer

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A Program that sets Standards.

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Editorial

Editorial

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Over the last years Urolithiasis has developed into a typical disease of civilisation. If a spontaneous discharge of the urinary stone is not possible, several therapy alternatives are available. The decision for the right therapy depends primarily on location, size and composition of the stone.

In addition to extracorporeal shockwave lithotripsy (ESWL) and percutaneous nephrolithotomy (PCNL), ureterorenoscopy (URS) is one of the most established methods of stone treatment. During this procedure, a stone basket with or without tip is usually applied following the localization of the stone fragments.

Due to the delicate elaboration of the UROMED STONIZER the application is ensured also in combination with a flexible ureterorenoscope. The rotatable basket and the flexible wire allow simple grasping of stones.

All UROMED STONIZER devices are made of highly elastic nitinol taut wires and are characterized by high shape stability even at strain. Due to the low risk of traumatisation, the UROMED stone retrieval baskets can be used even in difficult urinary tract anatomy.

The material for most flexible application

Nitinol, a highly elastic nickel-titanium-alloy, is eminently suitable for medical instruments. Because of its extreme thinness, flexibility and high kinking resistance it is perfect for stone extraction by ureterorenoscopy.

Concerning break resistance, flexibility and the low traumatisation risk, all UROMED STONIZER made of nitinol taut wire are far superior to conventional flat wire instruments made of steel. The nitinol taut wire features a high biocompatibility and has higher elastic deformability and also providing shape stability.

Due to these features, high functionality is provided also under high strain, e.g. when the flexible ureterorenoscope is being angled. Therefore, baskets made of nitinol became state-of-art within the last years.

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
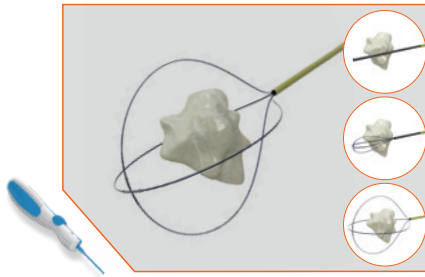
STONIZER stone retrieval basket, tipless, stick handle

S. 6-9

STONIZER Tube-in-Tube
REF 73201210

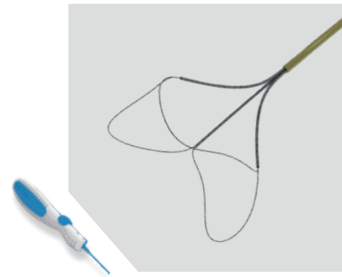4-Draht-Körbchen, gerade
4-wire basket, straight


STONIZER TWONE
REF 7215/20 ...

offener 3-Draht-Greifer
open 3-wire grasper

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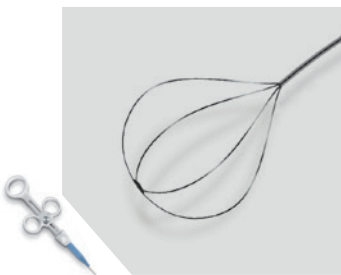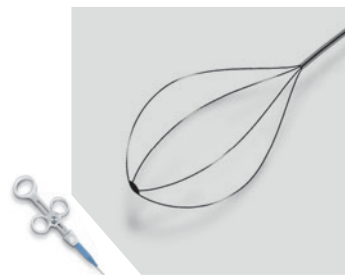
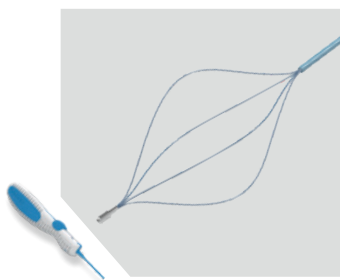
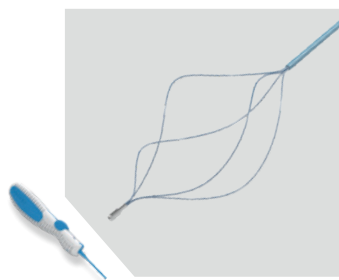
4-Draht-Körbchen, gerade
4-wire basket, straight


**STONIZER
REF 7530/40 ...**

4-Draht-Körbchen, helikal
4-wire basket, helical

STONIZER Steinfangkörbchen, mit Spitze und 3-Finger-Griff
STONIZER stone retrieval basket, with tip, 3-finger handle

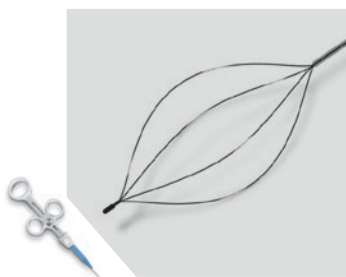S. 14-15

**STONIZER
REF 7825/30 ...**

4-Draht-Körbchen, gerade
4-wire basket, straight

**STONIZER
REF 7930/40 ...**

4-Draht-Körbchen, helikal
4-wire basket, helical

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*The new UROMED TONIZER Tube-in-Tube employs a novel technology: the black inner indicator tube is retracted into the golden outer tube during capture of the stone. A highly efficient operation is achieved in only three steps.
The innovative product design enables a reliable application, even when the product is bent around a tight angle.*

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Here is an overview of your user benefits:

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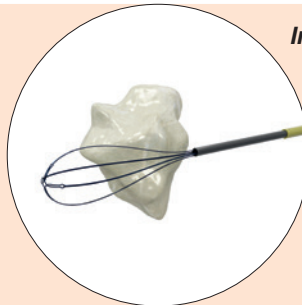
Orientierung - even when visibility is poor:

Even while it is still closed, the black indicator tube shows where the stone retrieval basket will be formed. This design highlight provides a reliable reference point even when visibility is limited.

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
Innovative Tube-in-Tube technology:

By retracting the black inner indicator tube the stone retrieval basket is formed while minimizing the risk of displacing the stone, thus ensuring reliable extraction of the stone fragments.

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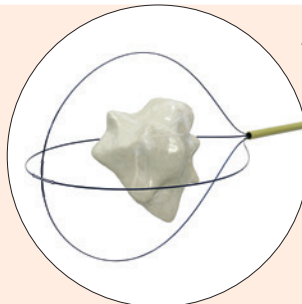
Efficiency due to technological reasons:

The black indicator tube has been completely retracted. The basket has formed completely and in precisely the correct position, as previously defined. So there is absolutely no need to correct the optics. Excellent 1:1 transmission of the rotation permits easy catch of the stone.

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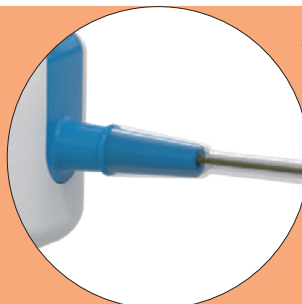Reliable application:

Due to a reinforced shaft and an optimal tube design, the friction is minimized so that the basket can be easily opened and closed, even when it is bent around a tight angle.

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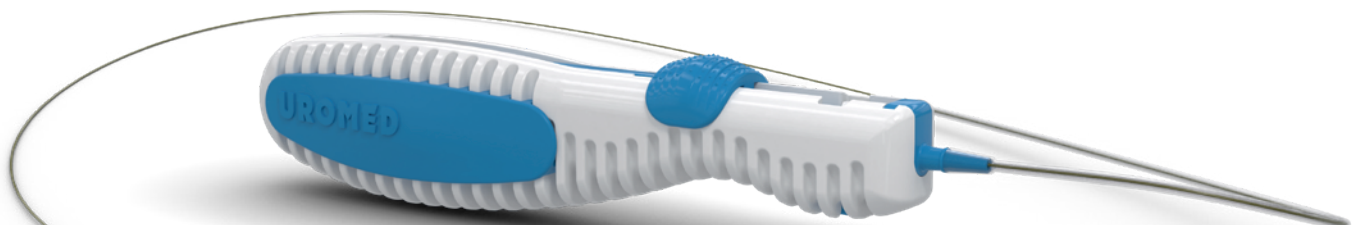
Retrieval basket for the urological removal of stones from the ureter and kidney during ureterorenoscopy

- *the tipless basket is suited for use in the kidney*
- *Tube-in-Tube technology minimizes changes in position during opening and closing*
- *the black indicator tube acts as a guide to the place where the basket will be formed later*
- *the reinforced shaft provides high functionality, even when it is bent around a tight angle*
- *the filigree basket design makes the basket optimally adapted to the conditions in the kidney*
- *using nitinol provides the basket element with high shape stability and opening force for use in the kidney*
- *the PTFE coating ensures the optimum sliding properties of the nitinol taut wire*
- *very effective control of the basket based on 1:1 transmission of rotation*
- *atraumatic distal end of the basket*
- *the insertion guide contributes to optimal insertion of the stone retrieval basket into the endoscope*
- *the ergonomic stick handle with thumb tab allows an intuitive one-handed use*
- *practical presentation of the basket in high-quality blister packaging with the additional clipless dispenser*
- *single use*

REF	Ch./Fr.	Ø Schaft	Länge	Ø Korb
REF	Ch./Fr.	Ø Shaft	Length	Ø Basket
73201210	2,0	0,6mm	1200mm	10mm

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Packaging: sterile
Original carton: 5 pieces

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More than just a Stone Retrieval Basket

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Ergonomic Stick Handle

The ergonomic stick handle allows one-handed operation by the intuitively operable thumb tab.

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Bend Protection
for high functionality.

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Stable Grasping Element

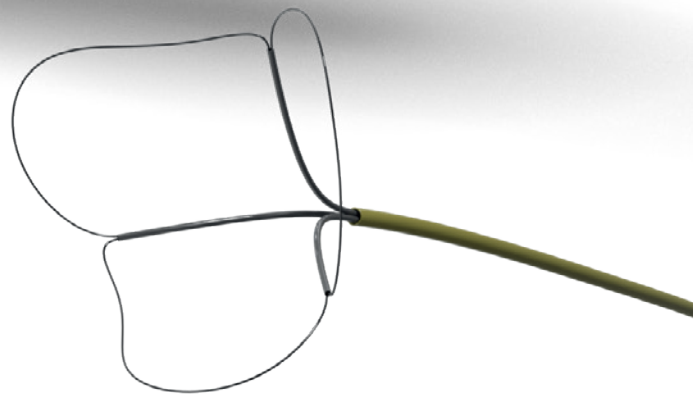
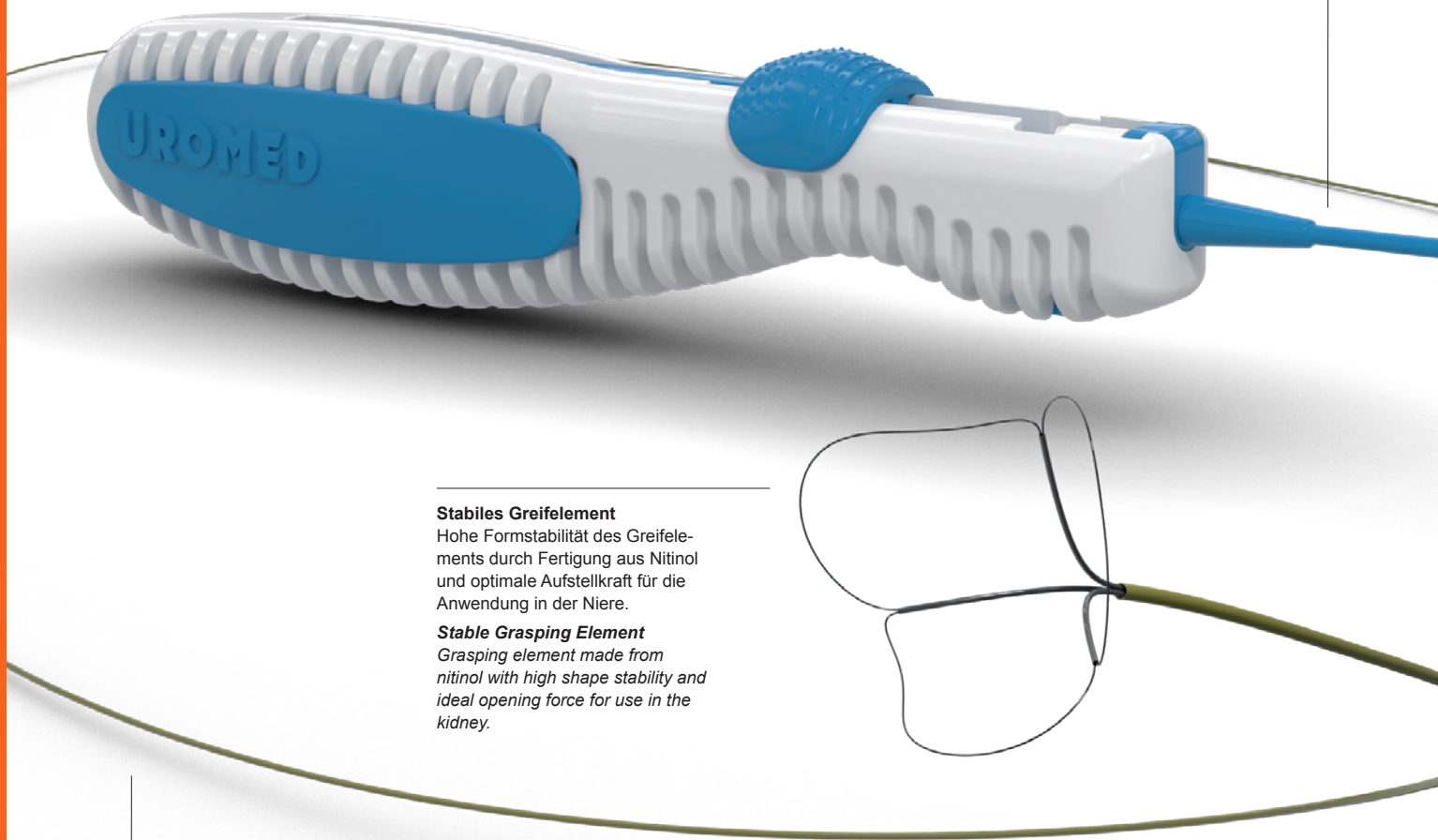
Grasping element made from nitinol with high shape stability and ideal opening force for use in the kidney.

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Flexible Shaft

The shaft is flexible and bend-proof, the coating of the taut wire allows a smooth intrusion.

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Only to be used with enclosed insertion guide!

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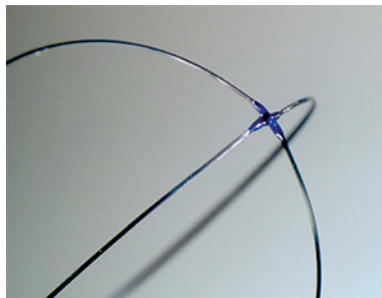
The STONIZER *advanced*

With circular basket design and loop technique.

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Retrieval basket for the urological removal of stones from the ureter and kidney during ureterorenoscopy

- *for use in kidneys*
- *high shape stability due to nitinol basket*
- *one-handed use*
- *mini basket: 15% less space required compared to REF 77191212 (basket length 17,5 mm)*
- *innovative construction of basket and modified materials ensure the highest possible reliability – also in case of stone streets*
- *pull and pressure stability of the basket because of its durable material*
- *mechanism to limit the tensile strength provides secure stone retrieval*
- *atraumatic distal end of basket*
- *guidance of the basket with high torque stability*
- *high functionality even with bended shaft*
- *flexible nitinol material*
- *increased flushing is possible due to the small outer diameter*
- *removable handle by knurled screw*
- *single use*
- *insertion guide*

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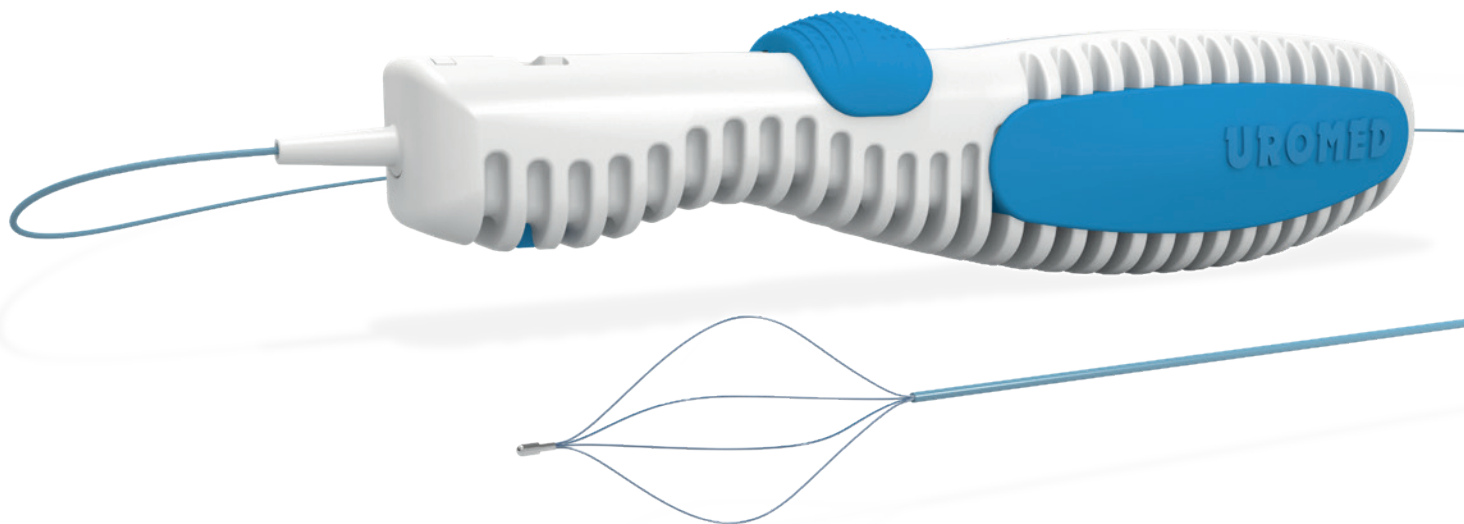
The **STONIZER** with tip is available with proven 3-finger handle or **stick handle**.

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Retrieval basket for the urological removal of stones from the ureter and kidney during ureterorenoscopy

- for use in the ureter
- high shape stability due to nitinol basket
- the ergonomic stick handle with thumb tab allows an intuitive one-handed use
- stepped tip feature guarantees concentric closing of the basket
- excellent gliding ability of the nitinol taut wire due to PTFE coating
- high functionality even with intense bended shaft
- excellent rotation characteristics (1:1 rotation)
- high-quality blister packaging with additional clipless dispenser
- single use

REF REF	Ch./Fr. Ch./Fr.	Ø Schaft Ø Shaft	Länge Length	Ø Korb Ø Basket
74259015	2,5	0,8 mm	900 mm	15 mm
74309015	3,0	1,0 mm	900 mm	15 mm

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©UROMED Kurt Drews KG