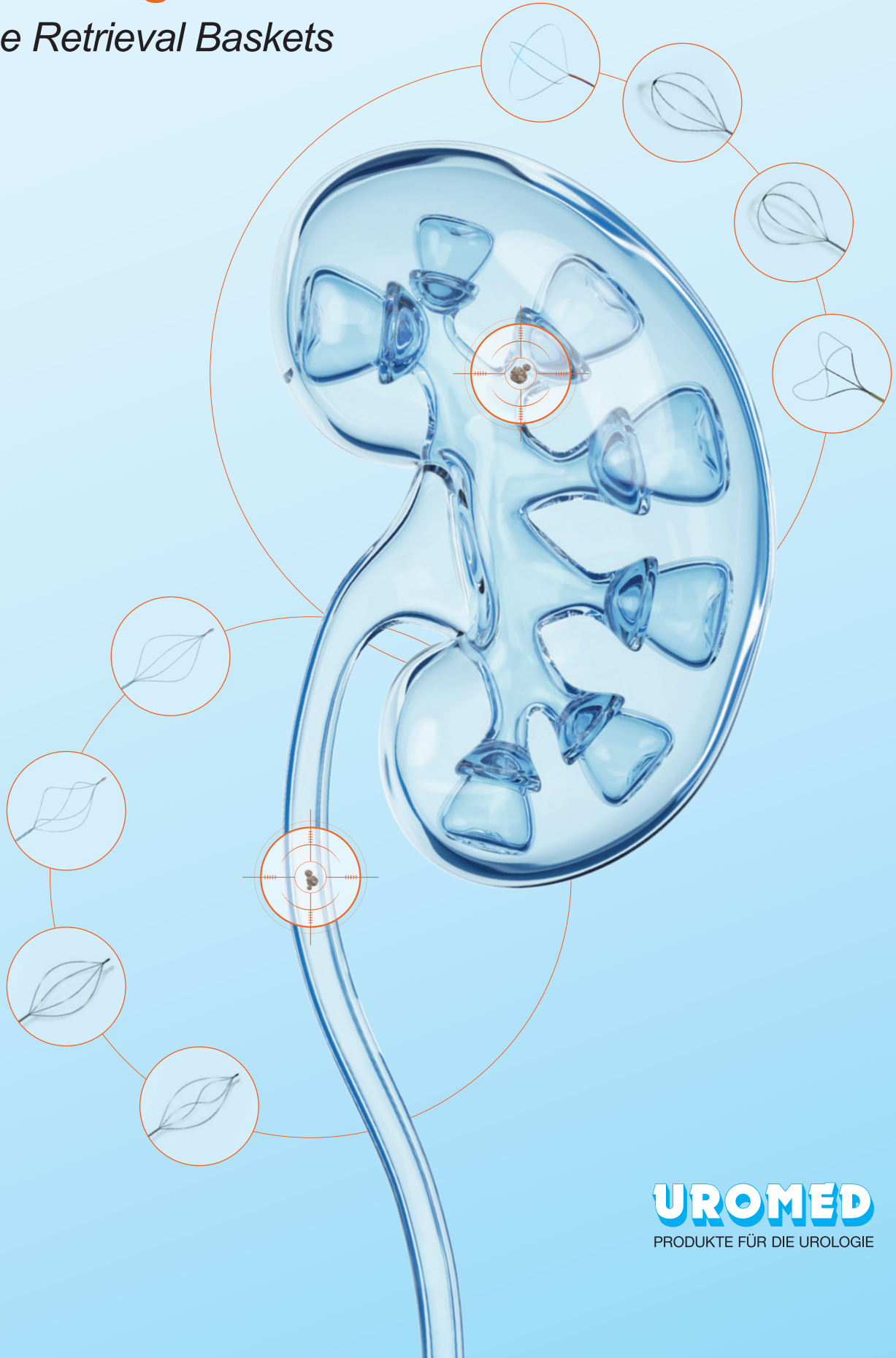
Steinfangkörbchen

Stone Retrieval Baskets

Editorial

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In recent years, urolithiasis has developed into a typical disease of civilisation. If spontaneous removal of the urinary stone is not possible, various treatment alternatives are available. The choice of treatment mainly depends on the location, size and composition of the stone.

The established procedures include extracorporeal shock wave therapy (ESWL), percutaneous nephrolithotomy (PCNL) and ureterorenoscopy (URS). In URS, a stone retrieval basket is usually inserted after localization of the stone fragments.

The UROMED STONIZER are specially designed for application with flexible ureterorenoscopes. The filigree processing of these instruments enables good rotational behaviour of the basket elements and facilitates stone retrieval.

The flexible wires of the STONIZER are made of highly elastic Nitinol, a nickel-titanium-alloy that is characterized by high shape stability and break resistance. They ensure a high level of functionality even under heavy loads, such as bending of the flexible ureterorenoscope.

Thanks to the low risk of traumatization, the UROMED stone retrieval baskets can even be used in difficult ureteral anatomies.

INHALT

CONTENT

UROMED STONIZER Steinfangkörbchen

UROMED STONIZER Stone Retrieval Baskets

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Program Overview

4

Spitzenlose Steinfangkörbchen

Tipless Stone Retrieval Baskets

UROMED STONIZER ROTAR³⁶⁰ Steinfangkörbchen

UROMED STONIZER ROTAR³⁶⁰ Stone Retrieval Basket

6

UROMED STONIZER TWONE Steinfänger

UROMED STONIZER TWONE Stone Extractor

8

UROMED STONIZER *advanced* Steinfangkörbchen

UROMED STONIZER advanced Stone Retrieval Basket

10

UROMED STONIZER Steinfangkörbchen

UROMED STONIZER Stone Retrieval Basket

11

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Stone Retrieval Baskets with Tip

UROMED STONIZER Steinfangkörbchen

UROMED STONIZER Stone Retrieval Basket

14

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Program Overview

spitzenlos, Magnetgriff

tipless, magnetic handle

Seite/Page 6

STONIZER ROTAR³⁶⁰ REF 7115/7118/7122...

4-Draht-Körbchen, gerade
4-wire basket, straight


spitzenlos, Stabgriff

tipless, stick handle

Seite/Page 8

STONIZER TWONE REF 7215/7220...

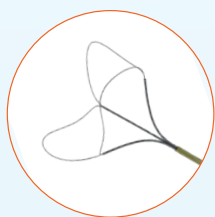offener 3-Draht-Greifer
open 3-wire grasper

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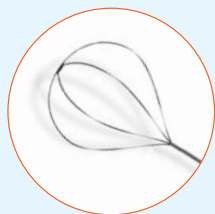
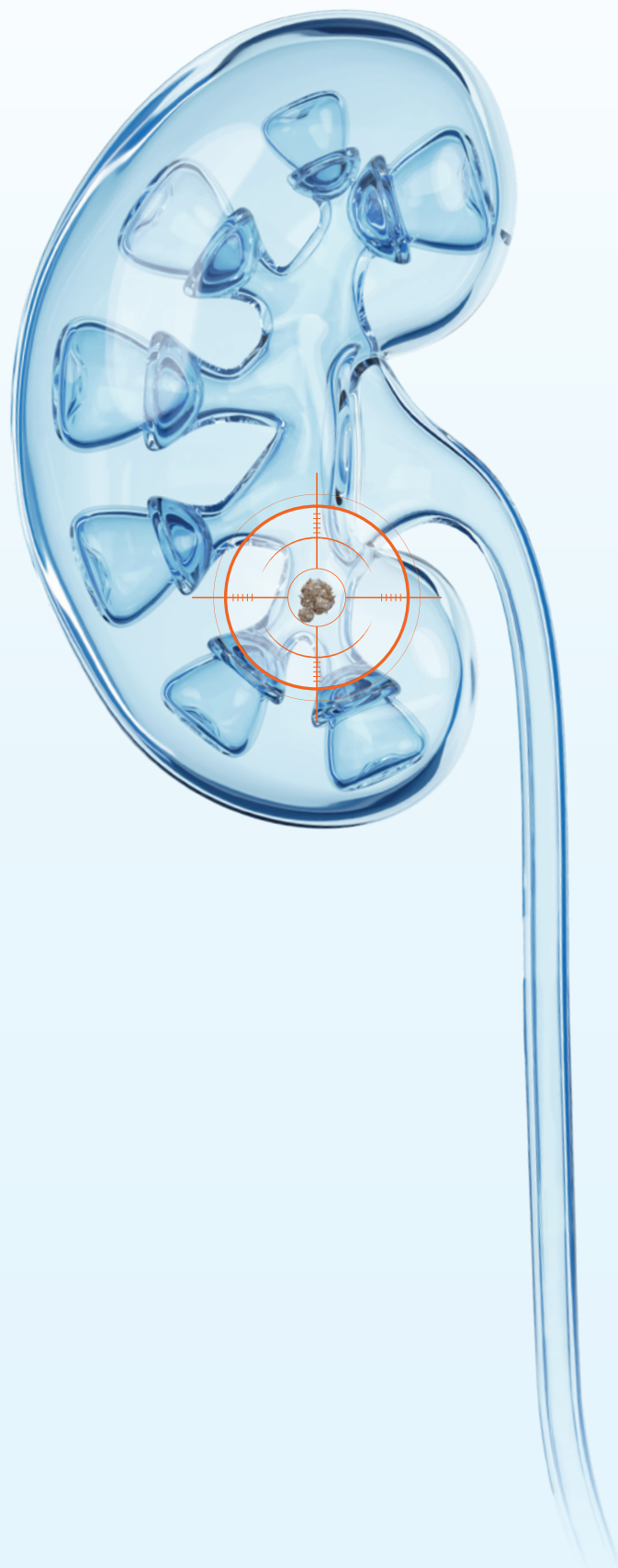
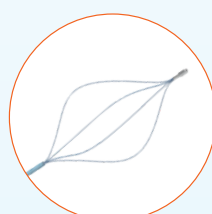
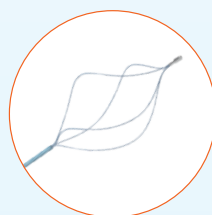
4-Draht-Körbchen, gerade
4-wire basket, straight

STONIZER
REF 7530/7540...

4-Draht-Körbchen, helikal
4-wire basket, helical

Seite/Page 15

mit Spitze, 3-Finger-Griff
with tip, 3-finger-handle

STONIZER
REF 7825/7830...

4-Draht-Körbchen, gerade
4-wire basket, straight

STONIZER
REF 7930/7940...

4-Draht-Körbchen, helikal
4-wire basket, helical

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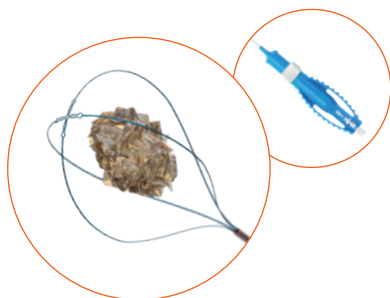
The new UROMED STONIZER ROTAR³⁶⁰ Stone Retrieval Basket with its patented magnet technology and ergonomic design sets new standards in the retrieval of kidney stones.

Eight high-quality neodymium magnets are used for precise control of the basket. If excessive tensile force is exerted, the magnets automatically and perceptibly decouple pull wire and thumb slider to ensure that the basket continues to function and to guarantee safe, controlled handling.

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Retrieval basket for the urological removal of stones during ureterorenoscopy

- for use in kidneys

Precision thanks to laser technology

- basket is cut from one piece
- improved visibility with commercially available ureterorenoscopes
- basket and shaft are firmly connected

Basket

- optimal radial force
- high durability and shape stability

Nitinol pull wire

- optimum gliding properties due to PTFE coating

Shaft

- kink-resistant and flexible
- small shaft diameter increases flushing flow in the working channel
- very precise rotation – 270° bendable shaft enables accessibility of all renal calyx groups

Handle

- ergonomically shaped and profiled – fits securely in the hand
- one-handed operation for left- and right-handers
- optimised handle length – also suitable for small hands
- lightweight construction

REF REF	Ch./Fr. Ch./Fr.	Ø Schaft Ø Shaft	Länge Length	Ø Korb Ø Basket
71151209	1,5	0,5 mm	1200 mm	9 mm
71151212	1,5	0,5 mm	1200 mm	12 mm
71181209	1,8	0,6 mm	1200 mm	9 mm
71181212	1,8	0,6 mm	1200 mm	12 mm
71181215	1,8	0,6 mm	1200 mm	15 mm
71221212	2,2	0,7 mm	1200 mm	12 mm
71221215	2,2	0,7 mm	1200 mm	15 mm

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Strain relief through magnet decoupling

- safeguards the function of the basket under a high degree of strain
- protects ureteral access sheath and fURS

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Magnetic 4D thumb slider

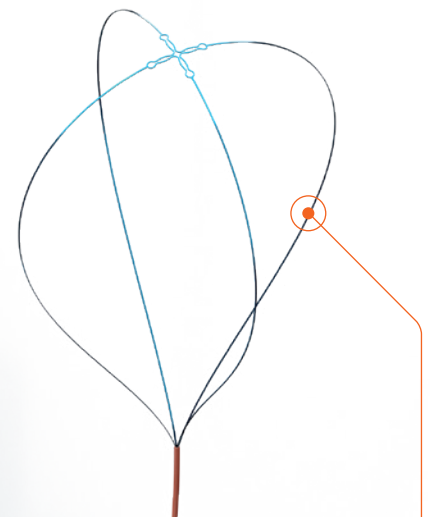
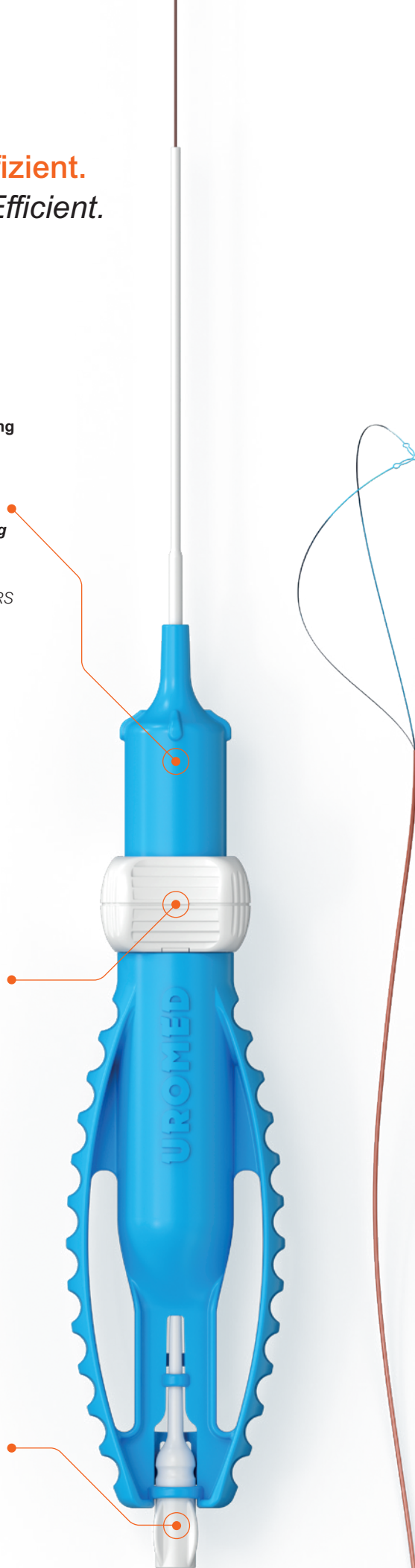
- combines opening, closing and rotation in a single movement
- 1:1 rotation behaviour – movement directly transferred to the basket
- stones are easily mobilised by 360° continuous rotation of the basket

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Insertion aid

- integrated in the handle for optimum handling
- long, narrow tip for firm positioning in the ureterorenoscope

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Flat wire construction

- effective grasping
- reliable fixation
- secure hold

UROMED STONIZER TWONE

Steinfänger

Stone Extractor

REF 7215...
REF 7220...

offener 3-Draht-Greifer
open 3-wire grasper

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The UROMED STONIZER TWONE Stone Extractor combines the advantages of an open grasper and a stone extractor in a single product.

The basket design makes it possible to grasp stones of different sizes, to safely remove them and, if necessary, to reposition or release them.

Only to be used with enclosed insertion guide!

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Retrieval basket for the urological removal of stones during ureterorenoscopy

- for use in kidneys

Basket

- optimal, frontal capture of stones due to open basket design
- high shape stability due to nitinol

Nitinol pull wire

- the PTFE coating ensures optimal sliding properties

Shaft

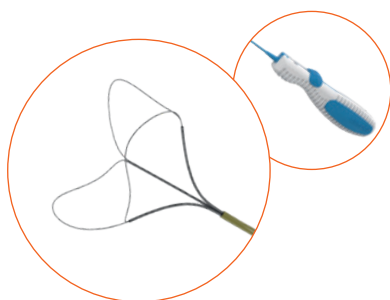
- high functionality even with bended shaft
- small shaft diameter increases flushing flow in the working channel

Handle

- the ergonomic stick handle with thumb tab allows an intuitive one-handed operation

Insertion guide

- assists insertion into commonly used ureterorenoscopes

REF REF	Ch./Fr. Ch./Fr.	Ø Schaft Ø Shaft	Länge Length	Ø Korb Ø Basket
72151208	1,5	0,5 mm	1200 mm	8 mm
72151211	1,5	0,5 mm	1200 mm	11 mm
72201208	2,0	0,6 mm	1200 mm	8 mm
72201211	2,0	0,6 mm	1200 mm	11 mm

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Packaging: sterile
Original carton: 5 pieces

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More than just a Stone Retrieval Basket

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Stable grasping element

- grasping element made from nitinol with high shape stability

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Ergonomic stick handle

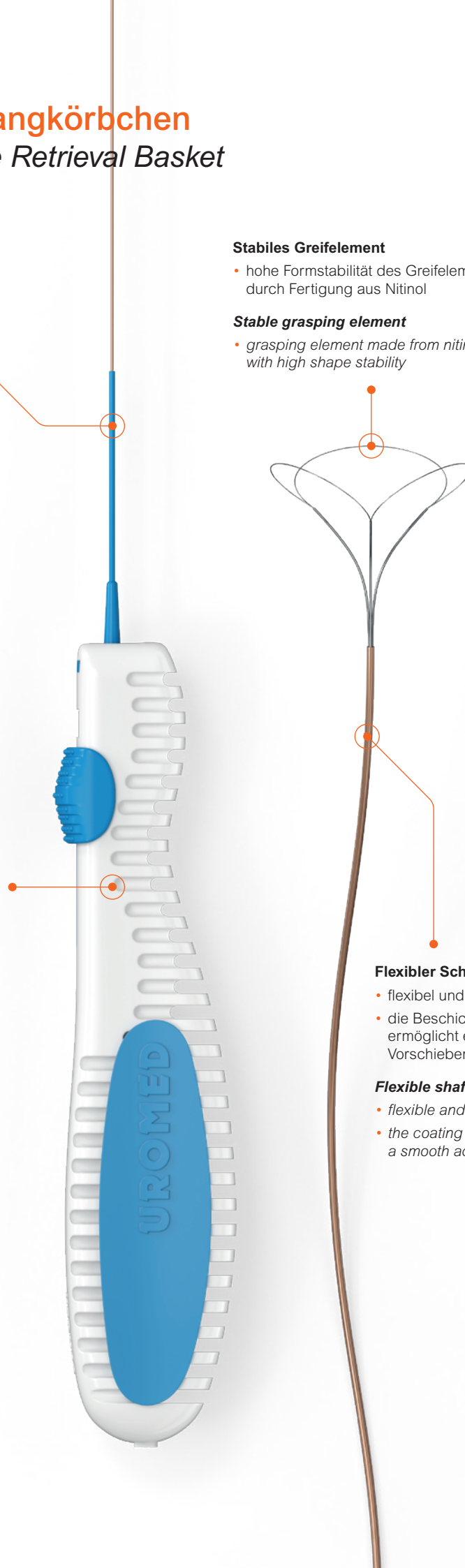
- the ergonomic stick handle allows one-handed operation by the intuitively operable thumb tab

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Flexible shaft

- flexible and kink-resistant
- the coating of the pull wire allows a smooth advancement

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The design of the basket and modified materials ensure maximum reliability – even when retrieving stone roads.

Mini basket: The circular basket shape and the loop technology enable a 15 % smaller space requirement compared to the REF 77191212.

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Retrieval basket for the urological removal of stones during ureterorenoscopy

- for use in kidneys

Basket

- high shape stability due to nitinol
- flat wire construction for effective grasping and secure hold
- atraumatic distal end of basket
- pull and pressure stability of the basket because of its durable material
- guidance of the basket with high torque stability

Nitinol pull wire

- mechanism to limit the tensile strength provides secure stone retrieval
- flexible nitinol material

Shaft

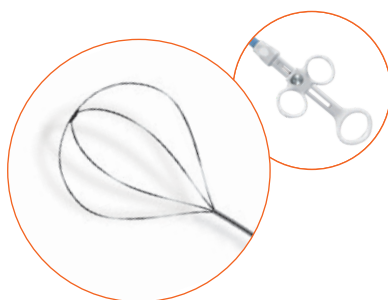
- high functionality even with bended shaft
- small shaft diameter increases flushing flow in the working channel

Handle

- intuitive one-handed operation with 3-finger-handle

Insertion guide

- assists insertion into commonly used ureterorenoscopes

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Universal standard basket
Robust and durable –
for reliable stone recovery.

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**Retrieval basket for the urological
removal of stones during
ureterorenoscopy**

- for use in kidneys

Basket

- high shape stability due to nitinol
- flat wire construction for effective grasping and secure hold
- atraumatic distal end of basket
- pull and pressure stability of the basket because of its durable material
- guidance of the basket with high torque stability

Nitinol pull wire

- mechanism to limit the tensile strength provides secure stone retrieval
- flexible nitinol material

Shaft

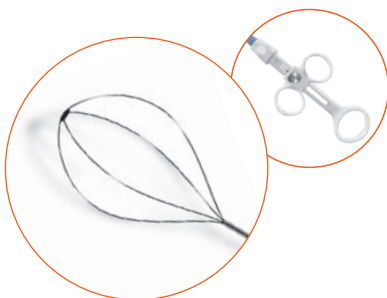
- high functionality even with bended shaft
- small shaft diameter increases flushing flow in the working channel

Handle

- intuitive one-handed operation with 3-finger-handle

Insertion guide

- assists insertion into commonly used ureterorenoscopes

REF REF	Ch./Fr. Ch./Fr.	Ø Schaft Ø Shaft	Länge Length	Ø Korb Ø Basket
77131208	1,3	0,4 mm	1200 mm	8 mm
77131212	1,3	0,4 mm	1200 mm	12 mm
77191208	1,9	0,6 mm	1200 mm	8 mm
77191212	1,9	0,6 mm	1200 mm	12 mm
77191216	1,9	0,6 mm	1200 mm	16 mm
77251212	2,5	0,8 mm	1200 mm	12 mm
77251216	2,5	0,8 mm	1200 mm	16 mm

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Minimal risk of injury

This tipless STONIZER is designed for use in the ureter, minimizing the risk of injury to the delicate ureter walls.

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Retrieval basket for the urological removal of stones during ureterorenoscopy

- for use in the ureter

Basket

- high shape stability due to nitinol
- flat wire construction for effective grasping and secure hold
- high opening force of the basket in the ureter
- atraumatic distal end of basket
- pull and pressure stability of the basket because of its durable material
- guidance of the basket with high torque stability

Nitinol pull wire

- mechanism to limit the tensile strength provides secure stone retrieval
- flexible nitinol material

Shaft

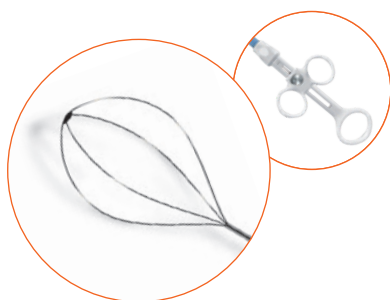
- increased flushing is possible due to the small outer diameter

Handle

- intuitive one-handed operation with 3-finger-handle

Insertion guide

- assists insertion into commonly used ureterorenoscopes

REF	Ch./Fr.	Ø Schaft	Länge	Ø Korb
REF	Ch./Fr.	Ø Shaft	Length	Ø Basket
77309016	3,0	1,0 mm	900 mm	16 mm

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This STONIZER with tip is known for its excellent rotation behaviour. The stepped tip element enables concentric closing of the basket on the shaft.

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Retrieval basket for the urological removal of stones during ureterorenoscopy

- for use in the ureter

Basket

- high shape stability due to nitinol
- high opening force of the basket in the ureter

Nitinol pull wire

- excellent gliding ability due to PTFE coating

Shaft

- high functionality even with intense bended shaft

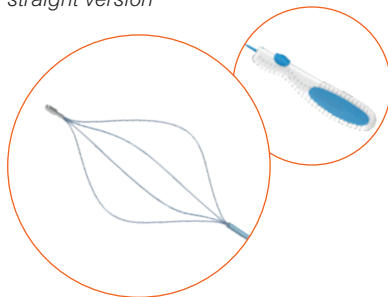
Handle

- the ergonomic stick handle with thumb tab allows an intuitive one-handed operation


Insertion guide

- assists insertion into commonly used ureterorenoscopes

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
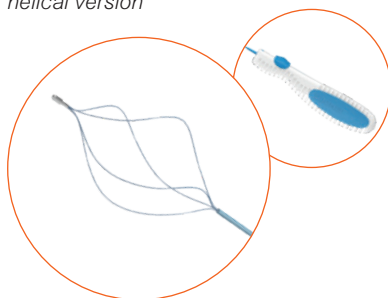
The durable material (Nitinol) ensures reliable tensile and compressive stability of the basket.

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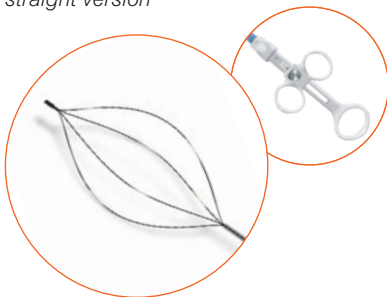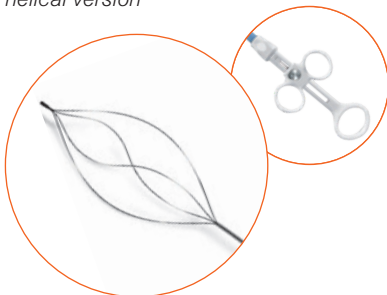Retrieval basket for the urological removal of stones during ureterorenoscopy

- for use in the ureter

Basket

- high shape stability due to nitinol
- high opening force of the basket in the ureter
- guidance of the basket with high torque stability

Nitinol pull wire

- mechanism to limit the tensile strength provides secure stone retrieval
- flexible nitinol material

Shaft

- high functionality even with bended shaft

Handle

- intuitive one-handed operation with 3-finger-handle

Insertion guide

- assists insertion into commonly used ureterorenoscopes

REF REF	Ch./Fr. Ch./Fr.	Ø Schaft Ø Shaft	Länge Length	Ø Korb Ø Basket	Ausführung Version
78259016	2,5	0,8 mm	900 mm	16 mm	gerade / straight
78309016	3,0	1,0 mm	900 mm	16 mm	gerade / straight
79309016	3,0	1,0 mm	900 mm	16 mm	helikal / helical
79409016	4,0	1,3 mm	900 mm	16 mm	helikal / helical

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*All indications correspond to our
knowledge at the date of press.
Due to technical progress all products
are subject to change.*

MA0016 · Rev. 5 · 01/2025

©UROMED Kurt Drews KG